

WARRANTY CARD	
Model No.:	Lot No.:
Invoice No.:	Date of Purchase:
Purchased By:	Contact No.:
Address:	
Dealer's Name:	
Register your product to claim warranty	
	
Dealer's Sign & Stamp	

* Terms & Conditions apply

Passim Lifesciences Ltd. warrants this units free of any defect in material, workmanship and operation, under normal use, for a period of one year from the date of purchase. If any part become defective during the warranty period Passim Lifesciences Ltd. will repair or replace (if not repairable) the same, free of cost. This shall not apply to any parts that are considered as expandible or deteriorable in the course of normal use. Passim Lifesciences Ltd. shall be relieved of any liability and warranty shall cease to apply if :

- This is not used in accordance with the instructions in the operational manual.
- It is used with any equipment not complying with the specification of this unit.
- It is not regularly maintained.
- The unit is disassembled, repaired or operated by person not authorized by Passim Lifesciences Ltd.
- Damage caused due to negligence.
- The unit is operated in corrosive materials or in the harmful atmosphere.
- The warranty card is not filled completely and produced at the time of warranty claim.

SYMBOL

 Medical Device  Manufactured By  Refer Instructions Manual  Lot No.

 Keep Away From Sunlight  Manufacturing Date  Serial No.  No Trash

 Do not use  Warning/Caution  Must not be exposed in rain or excessive moisture

Manufactured by:

Passim Lifesciences Ltd. Plot No. 45, Ind. Area, Phase-II,
Panchkula - 134113 (Haryana) - INDIA

File no.: Passim-Panch-HR/M/MD/013560

For any Complaint / Suggestion please contact

Customer care Number: 1800 309 3009

Timing: 9am-7pm, Mon.-Sat.

Email ID: customercare@drodin.in

Website: www.drodin.in

IM/OST/118-00



Stethoscope (Uno)

MODEL NO.: OST007

INSTRUCTION MANUAL



Please read instruction manual before use

Thank you for choosing Stethoscope Uno Model OST007

Please take a moment to review this manual to become familiar with the recommended practices for using and caring for your stethoscope. Read instructions carefully before using and save for future reference.

Intended Use: A stethoscope is used to detect and study heart, lung, stomach, and other sounds in adult humans, human fetuses, and animals. Using a stethoscope, the listener can hear normal and abnormal respiratory, cardiac, pleural, arterial, venous, uterine, fetal and intestinal sounds. Stethoscope is an acoustic medical device for auscultation, or listening to the internal sounds or heart beats of human body.

Description of Device: The chest piece of single head made up of aluminum. Headset frame are stainless steel material and ear knob are soft PVC material, PVC tube are normal U shape. This is used for listening low and High frequency sound of organ. Stethoscope with headsets is ergonomically pre-angled. The headset is anatomically adjustable for best fit, comfort, and sound transmission. A headset adjusted to a 15° forward angle is standard, and should be fit with the ear knob pointed away from you if it is held directly in front of you. The single head chest piece equipped with diaphragm for detecting high-frequency sounds, According to the auscultation need, simply turn the fully-rotational acoustic valve stem to select the proper side of the chest piece. Hold the chest piece firmly against the desired auscultation location on the patient's external thoracic region. Prior to each use, make sure all parts are properly connected.

Instructions for Use :

Adjust your stethoscope's earpiece: Place the Stethoscope in Your Ears. The stethoscope headset is angled to complement the anatomy of the typical ear canal and is designed to provide a comfortable, acoustically sealed fit. The ear knob should point in a forward direction as you insert them into your ears. It is important to make sure that the earpieces are facing forward and that they fit well. Otherwise, you might not be able to hear anything with your stethoscope. Make sure that the earpieces are facing forward. If you put them in backwards, you won't be able to hear anything. Make sure that the earpieces fit snugly and have a good seal to keep out ambient noise. If the ear pieces don't fit well, most stethoscopes have removable earpieces.

Check the earpiece tension on your stethoscope: In other words, make sure that the earpieces are close to your head but not too close. If your earpieces are too tight or too loose, readjust them. If the earpieces are too loose, you may not be able to hear anything. To tighten the tension, squeeze the headset near the earpieces. If the earpieces are too tight, they might hurt your ears and you might have a hard time using your stethoscope. To reduce the tension, pull the headset apart gently.

How to listen Sound

Step 1: Use your stethoscope in a quiet environment to ensure accurate auscultation. Background noise can interfere with the detection of body sounds; therefore, choose a calm and quiet area where heart, lung, and other body sounds can be heard clearly.

Step 2: Request for your patient to be appropriately disrobed. Having your stethoscope chest piece placed directly on your patient's chest or back will eliminate distortions and frictional noise from clothing. Use the stethoscope on bare skin to avoid picking up the sound of rustling fabric. If your patient is a man with chest hair, keep the stethoscope still to avoid any rustling sounds.

Step 3: Proper patient positioning is essential for accurate auscultation. To assess heart and abdominal sounds, ask the patient to lie comfortably in a supine position. For lung auscultation, the patient should be seated upright. Heart, lung, and bowel sounds may vary depending on the patient's position, such as sitting, standing, lying supine, or lying on one side. Therefore, examining the patient in different positions—including supine, seated, and left lateral recumbent positions—can help identify and distinguish normal and abnormal sounds more effectively by enhancing their audibility.

Warning and safety instruction:

- Avoid extreme heat, cold, solvents, and oils.
- The entire stethoscope can be wiped clean with alcohol or soapy water. It is a best practice to use a disposable wipe when cleaning the stethoscope to remove organic material.
- Ear Knob can be removed from the ear tubes for thorough cleaning.
- Do not immerse your stethoscope in any liquid or subject it to steam sterilization. If disinfection is required, the stethoscope may be wiped with a 70% isopropyl alcohol solution.
- Clean stethoscope thoroughly before each use.
- Use clean cotton cloth and isopropyl alcohol solution to cleanse all soiled parts.
- To remove the ear tips from the headset, pull firmly. To assemble the ear knob, push the small side of the ear knob firmly onto the ear tube until it snaps fully into place.
- Do not immerse your stethoscope in any liquid. Do not subject your stethoscope to any sterilization process. Avoid storing your stethoscope in extreme heat.
- Avoid prolonged exposure of acoustic tubing to excessive heat, cold, solvents, moisture and oils/lipids (secreted by human skin).
- Keep out of children's reach.
- Keep it in a light case or a bag.

General Cleaning & Care: Clean stethoscope thoroughly before each use. For chest piece, acoustic tubing, and ear tips, use clean cotton cloth and rubbing alcohol or isopropyl alcohol solution to cleanse all soiled parts. Avoid prolonged exposure of the acoustic tubing to excessive heat, cold, solvents, moisture, or oils/lipids (secreted by human skin).

Removing the Diaphragm and Cleaning the Chest piece: With the diaphragm side down, push the rim at one point with both thumbs and roll the rim off the edge of the chest piece. Remove the diaphragm from the rim and clean parts in wipe with alcohol. Wipe chest piece surfaces with alcohol or soapy water. Dry all parts and surfaces thoroughly before reassembly.