

WARRANTY CARD

Model No.:..... Lot No.:.....

Invoice No.:..... Date of Purchase:.....

Purchased By:..... Contact No.:.....

Address:.....

Dealer's Name:.....

Register your product
to claim warranty

* Terms & Conditions apply

Dealer's Sign & Stamp

Warranty

Passim Lifesciences Ltd. provides a limited 30 days warranty on this unit, applicable only to the internal heating element, against manufacturing defects from the date of purchase. During the warranty period, any defective parts will be repaired or replaced free of charge, **excluding consumable components** such as the nasal steamer & facial steamer, which are not covered once used.

Warranty is void if:

- The device is misused or not maintained per this manual.
- Unauthorized repairs or modifications are made.
- Operated in corrosive or harmful environments.
- The warranty card is not fully completed and presented upon claim.

Symbols

MD Medical Device	Refer Instructions Manual	Warning/Caution
No Trash	Manufactured By	Must not be exposed in rain or excessive moisture
LOT Lot No.	Keep Away From Sunlight	Manufacturing Date
SN Serial No.	Do not use If Package damaged	

Manufactured by:

Passim Lifesciences Ltd.
 Plot No. 45, Ind. Area, Phase-II, Panchkula - 134113, Haryana-INDIA
 File No.: Passim-Panch-HR/M/MD/013560
 For any Complaint / Suggestion please contact:
 Customercare Number: 1800 309 3009, Timing: 9am-7pm, Mon.-Sat.
 Email ID: customercare@frodin.in, Website: www.drodin.in
 IM/OSV/90-01



INSTRUCTION MANUAL

STEAM VAPORIZER

OSV104



Thank you

Thank you for choosing Dr. Odin. We hope this steam vaporizer becomes a helpful part of your wellness and skincare routine. Please read this manual carefully and keep it for future reference.

What It's Used For

The Dr. Odin Steam Vaporizer produces warm, soothing steam for both **health** and **beauty** needs:

Health Benefits:

- **Colds, Flu & Cough:** Steam helps loosen mucus in the nose and throat, providing relief from congestion.
- **Dryness:** Steam adds moisture to dry nasal passages, especially helpful if you've used antihistamines or nasal sprays.

Beauty & Skin-care:

- Use it as a **facial steamer** for a gentle, spa-like experience at home.
- Steam opens pores and improves blood circulation, promoting healthier-looking skin.

Product Highlights

- Shock-free, lightweight plastic body
- 4-in-1 design: Nasal steamer for adults & kids, facial steamer and steam inhaler dome
- Detachable parts for easy cleaning
- Easy water filling
- Long power cord
- Safe, single-layer water tank to prevent electric shocks

Specifications

Feature	Details
Power Supply	230V ~ 50Hz
Power Consumption	550 Watts
Steam Start Time	30 seconds to 1 minute
Body Material	100% shock-proof plastic
Safety	Single-layer tank

Water Level Indicator Yes (marked inside tank)

How It Works

This vaporizer has a container that holds tap water. Once switched on, the heating element brings the water to a boil, producing steam. As the steam rises, it increases humidity around you. The steam is clean—impurities and minerals stay behind in the tank.

How to Use It (Step-by-Step)

- 1. Set it up**
Place the vaporizer on a flat, stable surface. Fully unroll the power cord.
- 2. Open the tank**
Twist and remove the upper cap.
- 3. Fill with water**
Add **tap water** up to the maximum level.
 Do not overfill. Do **not** add any medication or oils.
If steam is low, add a **pinch of salt** to help.
- 4. Reassemble & Attach**
Reattach the upper cap and fit the desired attachment (nasal or facial) on top.
- 5. Plug it in**
Connect the cord to a 220V socket. Wait 30 sec. -1 min. for steam to begin.
- 6. Steam begins**
The heating element produces clean steam. You'll notice the air around becoming warm and moist.
- 7. Start therapy**
Drape a towel over the back of your head and lean slightly toward the steamer (but do **not** touch it).

Inhale the steam through your nose or mouth for **5–15 minutes**.

8. After use

Turn off the power and unplug the unit. Let it cool before touching, refilling, or storing.

Cleaning & Maintenance

After Each Use:

- Remove the upper cap.
- Empty leftover water.
- Clean the tank using plain water to remove any mineral deposits (normal depending on your tap water).

Storage Tips

- Store in a **cool, dry place**.
- Avoid direct sunlight or extreme heat/humidity.
- Keep unplugged when not in use.
- Always store out of reach of children.

Safety Precautions (Must Read)

- Do not use the device if it appears damaged.
- New units may have a plastic smell—clean with lukewarm water twice before first use.
- Do not use **RO water** – use **tap water only**.
- Never add inhalants, essential oils, or medicine into the water tank.
- Never operate with an **empty tank**.
- Never open the cap while the device is running—let it cool first.
- Steam is hot. Don't place your face too close to the nozzle. Keep your eyes closed and protect heat-sensitive areas.
- Keep out of reach of children. Children must **not** operate or play with the device.
- Never immerse the unit in water.
- Do not touch the power plug with wet hands.
- Disconnect the device from power after each use.

Common Issues & Troubleshooting

Why is it making noise (buzzing, crackling, humming)?

- Possible Causes:
 - Too much salt
 - Low water level
 - Dirty tank
- Fix:
Unplug → Let it cool → Empty tank → Rinse → Refill (with less/no salt)

Why is steam coming out too fast?

- You've likely added too much salt.
- Fix:
Unplug → Let it cool → Empty and rinse → Refill with clean tap water.
- If your water is hard, try using **50% tap + 50% distilled water** (no salt).

Why is steam too slow or not starting?

- Check if the unit is plugged in.
- Your water might have low mineral content.
- Fix:
Add just a **pinch of salt** after unplugging and cooling the unit.

Final Note

Use your Dr. Odin steam vaporizer regularly for better breathing, glowing skin, and a soothing experience—right at home. For your safety and product longevity, always follow the instructions and safety tips provided.